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Besoin d'aide ?

**03 80 447 457**


Read this manual carefully and comply with the instructions given in it.  
Read warnings and contra-indications carefully.

**It is recommended to fully charge the patch before first use**


Use only the cable supplied and a CE-certified mains charger with the following output parameters:  
5V and 0.3 - 2A (printed on the charger). **Do not use fast chargers.**

### IDENTIFICATION OF THE PATCH FACES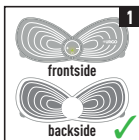


On a table, place the back face of the patch in front of you (control box against the table).

### PROGRAM SELECTION

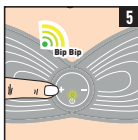
**3 programs** are available. **Each program lasts 30 minutes.** For more details, refer to instruction for use. When you turn on the patch **program 1 is selected by default**: this program is recommended for 1<sup>st</sup> use. **To change the program, press the + button for a long time** (approx. 3 seconds, until you hear a series of beep): 2 beeps indicate that program 2 is activated. **A further prolonged press** will switch to program 3: 3 beeps. Release the button. **To start the stimulation, go to the next step.**

### ADHESIVE GEL REFILL PREPARATION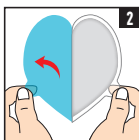


Open the bag and take one refill of adhesive gel. One refill corresponds to the right part of the patch and the other to the left part. Remove carefully the blue protective film, **making sure that the adhesive gel remains firmly in place on the transparent protective film.** **If necessary, you can retain the gel with your finger.** Keep the blue protective film, you will need it to put it back on the patch after each use.

### STARTING STIMULATION AND ADJUSTING THE INTENSITY

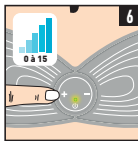Once you have selected your program, press only once and **briefly** on the + button to start stimulation. Adjust the intensity by briefly pressing on the + and - buttons. If the feeling is painful, decrease the intensity. **15 intensity levels** are available.

### APPLYING GELS ON THE PATCH

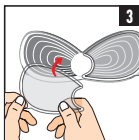Using the transparent protective film, place one of the gels on the patch correctly; making sure that **the tab of the gel is perfectly overlaid with the one of the patch.** Do the same with the 2nd gel by following again the steps 2 & 3. Once the two gels are positioned on the electrical circuits, remove the 2 transparent protective films making sure that the gel remains on the patch (if necessary, you can retain the gel with your fingers).

**1 pair of adhesive gels = about 30 uses**

*(Comment : you can throw away the transparent protective film)*

### SWITCHING OFF THE PATCH

**At the end of the 30 minutes, the patch shuts down automatically.**

If you want to switch off the patch before the end of the session of 30 minutes, press **briefly** on the button . 3 sound signals and 3 green light signals indicate that the patch is switched off.


### PATCH APPLICATION AND INITIATION

**Apply the patch on the painful area**, as shown on the schema, **press briefly on the button** . 1 sound signal is heard and a green light lights up. **This is program 1.**

If you do not see a green light or hear a beep, it means that the patch is completely discharged. Recharge the patch using the supplied cable.

### PATCH STORAGE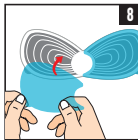

Before removing the patch, make sure it is switched off. **Then carefully** remove the patch, making sure that the gels remain on the electrical circuit (if necessary, you can retain the gel with your fingers). **Leave the adhesive gels on the patch and cover them with the preserved blue protective film.** Your patch is ready for the next use. For the next uses, remove the blue protective film and refer directly to step 4.

#### COMMENT:

**When the patch does not stick enough to the skin anymore, it is necessary to moisten the gels in order to extend their lifetime, as it is advised in the manual.**