GUIDE D'UTILISATION	<b>FR</b>	<b>PATCH URGO ELECTROTHERAPIE RÈGLES DOULOUREUSES RECHARGEABLE</b> .....	2
USER MANUEL	<b>EN</b>	<b>URGO ELECTROTHERAPY PAINFUL PERIODS RECHARGEABLE PATCH</b> .....	11
GEBRUIKERSHANDLEIDING	<b>NL</b>	<b>URGO NAVULBARE PATCH ELEKTROTHERAPIE PIJNLIJKE MENSTRUATIE</b> .....	20
GEBRAUCHSANWEISUNG	<b>DE</b>	<b>URGO WIEDERAUFLADBAREN TENS-PATCH FÜR REGELSCHMERZEN</b> .....	29
MANUAL DE USO	<b>ES</b>	<b>URGO PARCHE ELECTROTHERAPIA PARA MENSTRUACIONES DOLOROSAS RECARGABLE</b> .....	38
GUIA DE UTILIZAÇÃO	<b>PT</b>	<b>URGO EMPLASTRO DE ELETROTHERAPIA RECARREGÁVEL PARA CÓLICAS MENSTRUAIS</b> .....	47
GUIDA ALL'USO	<b>IT</b>	<b>URGO CEROTTO ELETTOTERAPIA MESTRUAZIONI DOLOROSE RICARICABILE</b> .....	56
INSTRUKCJA OBSŁUGI	<b>PL</b>	<b>URGO ELEKTROSTYMULATOR DO ŁADOWANIA NA BOLESNE MIESIĄCZKOWANIE</b> .....	65
NÁVOD K POUŽITÍ	<b>CZ</b>	<b>URGO ELEKTROTHERAPIE BOLESTIVÁ MENSTRUACE DOBÍJECÍ NÁPLAST</b> .....	74

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**JKH Health Co., Ltd.**  
Address: 4-5F, Building 12, Hengmingzhu Industrial Park, Xinqiao  
Tongfuyu Industrial Area, Shajing, Baoan, Shenzhen, CHINA

Shanghai International Holding Corp. GmbH (Europe)  
Eiffestrasse 80, 20537 Hamburg, GERMANY

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**CE 1639**


# QUICK START GUIDE BEFORE FIRST USE

## URGO ELECTROTHERAPY PAINFUL PERIODS RECHARGEABLE PATCH


Read this manual carefully and comply with the instructions given in it. Read warnings and contraindications carefully.

 **It is recommended to fully charge the patch before first use.**


Use only the cable supplied and a CE-certified mains charger with the following output parameters: 5V and 0.3 - 2 A (printed on the charger). **Do not use fast chargers.**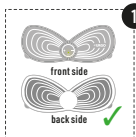


- 1 IDENTIFICATION OF THE PATCH FACES**  
On a table, place the back side of the patch in front of you (control box against the table).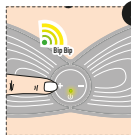


- 5 PROGRAM SELECTION**  
**3 programs** are available. Each program lasts **30 minutes**. For more details please refer to the manual. When you turn on the patch **PROGRAM 1** is selected by default: this program is recommended for 1<sup>st</sup> use. **To change the program, press the + button for a long time** (approx. 3 seconds, until you hear a series of beeps): 2 beeps indicate that program 2 is activated. Release the button. A further prolonged press will switch to program 3: 3 beeps.  
**To start the stimulation, go to the next step.**

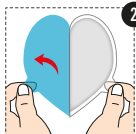- 2 ADHESIVE GEL REFILL PREPARATION**  
Open the bag and take a refill of adhesive gel. One refill corresponds to the right part of the patch and the other to the left part. Remove carefully the blue protective film, **making sure that the adhesive gel remains firmly in place on the transparent protective film**.  
If necessary, you can retain the gel with your finger. Keep the blue protective film you will need it to put it back on the patch after each use.

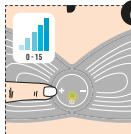
- 6 STARTING STIMULATION AND ADJUSTING THE INTENSITY**  
**Once you have selected your program, press only once and briefly on the + button to start stimulation.** Adjust the intensity by briefly pressing on the + and - buttons. If the feeling is uncomfortable or painful, decrease the intensity.  
**15 intensity levels are available.**

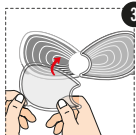
- 3 APPLYING GELS ON THE PATCH**  
Using the transparent protective film, place one of the gels on the patch correctly; making sure that **the tab of the gel is perfectly overlaid with the one of the patch**. Do the same with the 2<sup>nd</sup> gel by following again the steps 2 & 3.  
Once the two gels are positioned on the electrical circuits, remove the 2 transparent protective films making sure that the gel remains on the patch (if necessary, you can retain the gel with your fingers). **1 pair of adhesive gels = about 30 uses.**  
*(Comment : you can throw away the transparent protective film.)*

- 7 SWITCHING OFF THE PATCH**  
**At the end of the 30 minutes, the patch shuts down automatically.**  
If you want to switch off before the end of the session of 30 minutes, press briefly on the button . 3 sound signals and 3 green light signals indicate that the patch is switched off.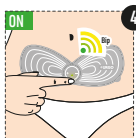

- 4 PATCH APPLICATION AND INITIATION**  
Apply the patch on the painful area, as shown in the illustration, and briefly press the button . 1 sound signal is heard and a green light lights up. **This is PROGRAM 1.**  
If you do not see a green light or hear a beep, it means that the patch is completely discharged. Recharge the patch using the supplied cable.

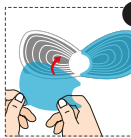- 8 PATCH STORAGE**  
Before removing the patch, make sure it is switched off. Then **carefully** remove the patch, making sure that the gels remain on the electrical circuit (if necessary, you can retain the gel with your fingers). **Leave the adhesive gels on the patch and cover them with the preserved blue protective film.** Your patch is ready for the next use. Store the patch in the transportation box provided.  
For the next uses, remove the blue protective film and refer directly to step 4.

See the patch's setting up on the website:  
[urgo.com](http://urgo.com) or via the QR code

**COMMENT:** When the patch does not stick enough to the skin anymore, it is necessary to moisten the gels in order to extend their lifetime, as it is advised in the manual.

## URGO ELECTROTHERAPY PAINFUL PERIODS RECHARGEABLE PATCH

EN

### SUMMARY

#### CONTENTS IN THE PACKAGE

1. PRESENTATION OF URGO ELECTROTHERAPY PAINFUL PERIODS RECHARGEABLE PATCH
2. PREPARATION BEFORE USE
3. USE
4. CONTRAINDICATED APPLICATION ZONES/ MEDICAL CONTRAINDICATIONS / PRECAUTIONS FOR USE
5. CLEANING
6. STORAGE
7. QUESTIONS & ANSWERS
8. DISPOSAL
9. TECHNICAL SPECIFICATIONS
10. WARRANTY

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

#### Contents in the package:

1 Rechargeable electro-stimulator

1 USB charging cable

1 Manual

1 Transportation box


2 Pairs of adhesive gels\*

\* 1 adhesive gel: about 30 uses depending on skin type/pilosity


■ **Electro-stimulateur:** The electro-stimulator (the patch) is made of silicone rubber, thin and flexible.

Control of programs and of intensity level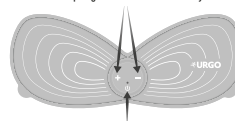

ON/OFF button

FRONT SIDE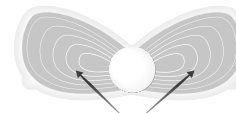


Electrical system


BACK SIDE

■ **Adhesive gel:** The adhesive surface is composed of a conductive hydrogel.

Adhesive gel composed of a conductive hydrogel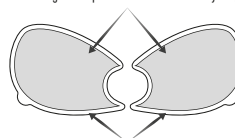

Protection film

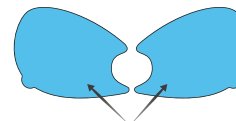Protection film

You can buy these conductive hydrogel in pharmacy.

Be sure that the conductive hydrogel you buy are compatible with the URGO ELECTROTHERAPY PAINFUL PERIODS RECHARGEABLE PATCH.

■ **The electrode:** The electrical system and the hydrogel constitute together the electrode.

■ **Charging USB cable:**

Before any request, contact the Customer Service Department:

Contact in FRANCE:

Laboratoires URGO HEALTHCARE - Customer Service Department

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21300 Chenôve - FRANCE

03 80 447 457 - email: contact@urgo.fr

## 1. PRESENTATION OF URGO ELECTROTHERAPY PAINFUL PERIODS RECHARGEABLE PATCH

URGO ELECTROTHERAPY PAINFUL PERIODS RECHARGEABLE PATCH works thanks to TENS Technology.

### 1.1 What is TENS technology and what does it consist of?

TENS (Transcutaneous Electrical Nerve Stimulation) technology provides **pain relief** using an appropriate intensity electric current. It is a **non-drug method** daily **used in pain centers**.

Whatever the pathology, TENS Technology does not have the power to cure the cause of pain but to reduce or eliminate the sensation of pain.

The electrical current is transmitted by electrodes (the electrical system and the hydrogel together) placed on the surface of the skin. It creates an electrical excitation of the nerves. Pain is reduced through two **actions**:

- **IMMEDIATE**: The reduction of pain transmission to the spinal cord, in other words: "gate control" effect.
- **LASTING**: The increase in the secretion of endorphins, natural hormones secreted by the brain.

Treatment with TENS technology provides an immediate and lasting pain relief.

**All TENS programs are efficient: this method has been scientifically tested and approved by the medical profession.**

### 1.2 Indications

**Before using electrotherapy, it is necessary that a health professional has diagnosed the origin and reason of your pain. Your pharmacist or doctor can give you instructions on how to perform electrotherapy self-treatment.**

Be sure not to use this patch on an unauthorized area, see chapter 4.1. Apply the device only on healthy, clean skin, without cream or greasy lotion.

It is recommended that the usage of the patch by mentally or physically impaired people takes place under the supervision of a responsible adult.

URGO ELECTROTHERAPY PAINFUL PERIODS RECHARGEABLE PATCH is indicated for chronic pelvic (lower abdominal) pain (e.g. endometriosis (program 3)) and for primary dysmenorrhea, i.e. menstrual pain that is not caused by any underlying gynecological disorder (programms 1 and 2). The pain begins before or at the beginning of the menstrual period, and most of the time it last from one to three days and it is manifested by cramps in the lower abdomen, which sometimes extend to the lumbar region.

### 1.3 How to choose your program and intensity level?

3 programs are available and each has 15 intensity levels. Each program lasts 30 minutes. The patch automatically turns off at the end of a program.

You should increase the intensity until the feeling of electrical stimulation provided by the patch is felt strongly without ever becoming painful. If the stimulation becomes painful, the intensity should be reduced. The program that you will choose should not provoke you:

- Painful sensations during the session.
- Significant muscle contractions (if the patch causes contractions, decrease the intensity and, if this is not enough, turn off the patch and reposition it by moving a few centimeters).
- Sensations of electric burns.

Each program can be set at intensity between 1 and 15. Always start the stimulation at a low intensity and gradually increase it until you **find the one that will give you the best relief**. Depending on the program used, this intensity may vary. The nervous system is very quickly accustomed to the current. During the first 5 minutes of a program or after several uses you may find that your sensitivity to stimulation decreases, and this is completely normal. Do not hesitate to increase the intensity to find a stronger stimulation level, but it has to stay painless.

- The perception of electrical stimulation is different for each person. **It is therefore recommended to TRY EACH PROGRAM in order to identify the one that offers you the best relief and comfort.** Depending on the type of pain you want to relieve with the electrotherapy patch, the program that works best for you may be different.

### Description of the 3 Electrotherapy Patch programs:

Program	1.	2.	3.
<b>Mode</b>	Combined.	Conventional continuous.	Conventional discontinuous.
<b>Benefits</b>	<ul style="list-style-type: none"> <li>• Quick and lasting pain relief.</li> <li>• Avoids the accoutumance.</li> </ul>	Quick pain relief.	Quick pain relief.
<b>Effects felt</b>	Alternation of vivid tingling sensation and pulsations associated with muscle micro-contractions.	Sensation of vivid tingling.	Sensation of vivid tingling.
<b>How does it work ?</b>	This program uses the Conventional TENS Mode at different frequencies (60 Hz, 80 Hz et 100 Hz) in continuous and discontinuous mode. The high-frequency intermittent stimulation with a fixed periodicity (2 Hz) is a Burst Mode that simultaneously activates 2 mechanisms of pain relief: gate control and release of endorphins.	It is a high frequency electrical stimulation (100 Hz) that directly excites the sensory nerves, thus blocking the transmission of the "pain" signal to the brain. Pain is relieved thanks to the so-called "gate control" effect.	It is a high frequency electrical stimulation (85 Hz) that directly excites the sensory nerves, thus blocking the transmission of the "pain" signal to the brain. Pain is relieved thanks to the so-called "gate control" effect.
<b>Duration (min)</b>	30	30	30

Program	1.	2.	3.
Current Type		Bidirectional.	
Frequency (Hz) <sup>1</sup>	Phase 1: 100 Hz Phase 3: 80 Hz Phase 5: 60 Hz	Phase 2: 100 Hz Phase 4: 80 Hz Phase 6: 60 Hz	100
Pulse duration, (µs) <sup>2</sup>	100	100	75
Mode <sup>3</sup>	DISCONTINUOUS & BURST The 3 frequency use (60 Hz, 80Hz and 100Hz) in the discontinuous mode & burst mode allows creating 6 different stimulation phases that last 1 minute each and are alternating during the program. Short pauses (1 s) are provided at the end of each phase to make stimulation more comfortable and enjoyable. <b>Phase 1:</b> DISCONTINUOUS (5 s ON and 1 s OFF) <b>Phase 2:</b> BURST 2 Hz (Work phase = 0,25 s; Pause phase = 0,25 s) <b>Phase 3:</b> DISCONTINUOUS (5 s ON and 1 s OFF) <b>Phase 4:</b> BURST 2 Hz (Work phase = 0,25 s; Pause phase = 0,25 s) <b>Phase 5:</b> DISCONTINUOUS (5 s ON et 1 s OFF) <b>Phase 6:</b> BURST 2 Hz (Work phase = 0,25 s; Pause phase = 0,25 s) Total duration of each phase = 1 min Work time = 59 s; pause time 1 s.	CONTINUOUS (no cycles)	DISCONTINUOUS (3 s ON and 1 s OFF)
Graph			
Battery life <sup>4</sup>	5 hours.	4 hours.	3 hours.

<sup>1</sup>Frequency (F) - measured in Hertz (Hz). This is the indicator that tells you how many times per second the stimulation will be performed. For example if frequency is 2 Hz, this means that you will be stimulated 2 times per second.

<sup>2</sup>Pulse duration (L) - measured in microseconds (µs). This is the duration of each stimulus, expressed in microseconds (µs). Each second has one million of microseconds in it.

<sup>3</sup>The mode can be CONTINUOUS or DISCONTINUOUS. CONTINUOUS: it is an uninterrupted stimulation. DISCONTINUOUS: the program includes short breaks between cycles to make the stimulation more comfortable and pleasant. To know how to run the different programs and adjust the intensity level, please, see section 3 "USE".

<sup>4</sup>Use at the highest intensity level.

### Usage advices:

If you wish, you can walk and go about your daily business while wearing the URGO Electrotherapy Painful Periods Rechargeable patch. As long as the patch remains well stuck, this should not pose a problem.

It is up to you to choose the program (s) that best suit you in terms of efficiency and comfort. Indeed, multiple factors will influence the sensory perception of the program (area concerned, sensitivity of your skin, fat mass, level of hairiness on the area to be treated, atmospheric humidity, etc.). The preferences of the same person can also change from one day to another.

For severe pain that requires fast relief, use by preference PROGRAMS 2 & 3 (conventional) as the first intention.

When the pain is less severe and to achieve lasting relief, use by preference PROGRAM 1.

For chronic pelvic pain (e.g. endometriosis) preferably use PROGRAM 3.

**Number of sessions:** as much as needed using the program that gives you the best relief, taking care to remove the patch between each session to let your skin rest and spacing the sessions by half an hour if possible.

In case of skin irritation, stop the treatment immediately and let your skin rest. If skin irritation persists and if you experience itching or irritation, consult your doctor before resuming use.

## 2. PREPARATION BEFORE USE

### BEFORE EACH USE:

- Be sure to apply the patch in accordance with the contraindications in section 4.
- Ensure that your skin is clean and free of creams or oils. Only in this way the patch can work optimally.
- Do not apply the patch on top of clothes.
- For an optimal effect, the patch electrodes must be in full contact with the skin.
- Make sure the patch is charged before starting the session: briefly press the "⏻" button: you will hear a "beep" sound and green light will light up. If you do not hear the sound neither see the green light, it means that your patch is completely discharged.
- Do not use the device while it is charging.

### PATCH CHARGING:


- The green light of the device will flash during charging and remain lit continuously once charging is complete.
- It is recommended to fully charge the patch before the first use.
  - Use only the cable supplied.
  - Use a suitable power supply: an electrical socket or a USB port (for example, that of a laptop or tablet). When charging from an electrical socket, use only a CE-certified mains charger for domestic use with the following output parameters: 5.0 V and 0.3-2 A (printed on the charger).
  - Do not use fast chargers: these types of chargers are incompatible with the patch and may damage the battery.
- Do not touch the electrodes during charging.
- This device contains a lithium battery. If the unit overheats during charging, stop charging and inform the distributor.

### TO PRESERVE THE BATTERY:

- The patch must be charged and discharged at least once every 3 months.

Find our manual in video on [www.urgo.com](http://www.urgo.com)


### 3.1 Starting with URGO ELECTROTHERAPY PAINFUL PERIODS RECHARGEABLE PATCH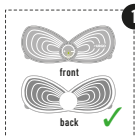

1

#### IDENTIFICATION OF THE PATCH FACES

On a table, place the back side of the patch in front of you (control box against the table).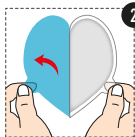

2

#### ADHESIVE GEL REFILL PREPARATION


For the application of the gel, the patch must be turned off.

Take out the two adhesive gel refills by opening the bag using the notch provided for this purpose, and without using scissors.

Take a refill of adhesive gel: one refill corresponds to the right part of the patch and the other to the left part. Remove carefully the blue protective film, **making sure that the adhesive gel remains firmly in place on the transparent protective film.** (If necessary, you can retain the gel with your finger.)

Make sure the adhesive gel does not have any defects (missing gel part, damaged gel, etc.) as this may cause skin irritation during use.

**Keep the blue protective film you will need it to put it back on the patch after each use.**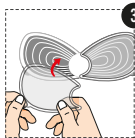

3

#### APPLYING GELS ON THE PATCH


Using the transparent protective film, place one of the gels on the patch correctly; making sure that the **tab of the gel is perfectly overlaid with the one of the patch.** Do the same with the 2<sup>nd</sup> gel by following again the steps 2 & 3.

Once the two gels are positioned on the electrical circuits, remove the 2 transparent protective films making sure that the gel remains on the patch (if necessary, you can retain the gel with your fingers). **1 pair of adhesive gels = about 30 uses**

(Comment : you can throw away the transparent protective film.)

#### PATCH APPLICATION


**Apply the patch on the painful area.** Make sure your skin is healthy and **has not been treated with creams or lotions** and is not particularly hairy. Press firmly to ensure good skin contact.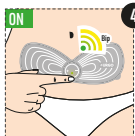

4

#### PATCH INITIATION

**To turn on the patch, briefly press the button "⏻":** 1 sound signal is heard and a green light lights up. **This is PROGRAM 1.** If you do not see a green light or hear a beep, it means that the patch is completely discharged. Recharge the patch using the supplied cable.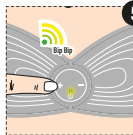

5

#### PROGRAM SELECTION


**3 programs** are available. Each program lasts **30 minutes**. For more details, refer to section 1.3.

When you turn on the patch **PROGRAM 1** is selected by default: this program is recommended for 1<sup>st</sup> use.

**To change the program, press the "+" button for a long time** (approx. 3 seconds, until you hear a series of beep): 2 beeps indicate that **PROGRAM 2** is activated. Release the button. **A further prolonged press** will switch to program 3: 3 beeps.

To return to the previous program press and hold the "-" button (about 3 seconds) until you will hear a series of beeps. **To start the stimulation, go to the next step.**

1 beep: Combined Program // 2 beeps: Conventional continuous Program // 3 beeps: Conventional discontinuous Program.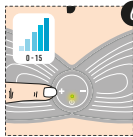

6


#### STARTING STIMULATION AND ADJUSTING THE INTENSITY

**Once you have selected your program, press only once and briefly on the "+" button to start stimulation.** Adjust the intensity by briefly pressing on the "+" and "-" buttons.

The adjustment of the intensity is individual and depends on the area to treat. Gradually increase the intensity until you feel a sensation of electrical stimulation that is well pronounced but not painful. If the feeling is uncomfortable or painful, decrease the intensity. Indeed, increasing stimulation intensity to a level that is too high can trigger pain and muscle contractions that are undesirable and do not bring additional efficiency. **15 intensity levels are available.**

Once the intensity has reached the highest or lowest level, the LED light and beep sound are emitted twice instead of once.

① If you hear a series of beeps during the session while you didn't touch the patch and in the same time the patch stops working, it means that the patch is not sufficiently in contact with the skin. The patch must be reapplied in order that the electrodes are fully in contact with the skin and you should restart the running program by briefly pressing the "+" button. The program will resume where it stopped but you will have to adjust the intensity level again.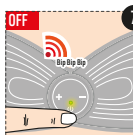


7

#### SWITCHING OFF THE PATCH

**At the end of the 30 minutes, the patch shuts down automatically.**

If you want to switch off before the end of the session of 30 minutes, press **briefly** on the button "⏻". 3 sound signals and 3 green light signals indicate that the patch is switched off.

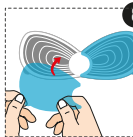8

#### PATCH STORAGE

Before removing the patch, make sure it is switched off. Then carefully remove the patch, making sure that the gels remain on the electrical circuit (if necessary, you can retain the gel with your fingers). **Leave the adhesive gels on the patch and cover them with the preserved blue protective film.** Your patch is ready for the next use. Store the patch in the transportation box provided.

For the next uses, remove the blue protective film and refer directly to STEP 4.

### 3.2 Maintenance and replacement of adhesive gels

#### ❗ Extending gel pad's lifetime

When the patch does not stick enough to the skin anymore, it is necessary to moisten the gels in order to extend their lifetime. Put a few drops of water on the entire surface of the gels but without wetting the central control box.

- Be careful not to leave some debris on the top of the gel (do not use a paper tissue).
- Wait until all the water has been absorbed by the gel before using the patch again.

#### ❗ Replacing the gel pad

- One gel allows about 30 uses.
- To change the adhesive gel, roll it over itself with your fingers.
- Check that there are no residues left on the electrical circuits before repositioning a new adhesive gel.
- Use this device only with compatible adhesive gels with URGO Electrotherapy Painful Periods Rechargeable patch. If any doubt, ask your pharmacist for advice.

### 3.3. Where to use the URGO ELECTROTHERAPY PAINFUL PERIODS RECHARGEABLE PATCH?

#### Abdominal pain

The patch must directly be placed on the painful area: low belly, down the belly button.

#### Low back pain

In the case of low back pain, make sure to position the electrodes from both sides of spine, so that the control box is placed on the spine.

For some locations, particularly for back pains, you may need the help of a third person.

## 4. CONTRAINDICATED APPLICATION ZONES / MEDICAL CONTRAINDICATIONS / PRECAUTIONS FOR USE

The device is not a substitute for medical consultation or treatment. Consult your pharmacist or your doctor first if you have any doubt.


### ⚠️ 4.1 Contraindications application zones

- Do not apply near the heart or torso as electrical signals may interfere with electrical signals from the heart.
- Do not apply in front of or on the sides of the neck (carotid, throat) because electrostimulation could trigger laryngeal and pharyngeal spasms leading to breathing problems. Stimulation of the carotid artery can cause a slow heart rate or unusually low blood pressure in people with blood vessel problems.
- This patch is not designed for an application on the head.
- Do not use on the genitals.

■ To sum-up do not use the patch on the following areas:

### 4.2 Medical contraindications

- Do not use if you have an implanted device (pacemaker, insulin pump, glucose meter, implanted defibrillator...), as URGO Electrotherapy Painful Periods Rechargeable patch may interfere with the operation of these devices.
- Do not apply on **skin lesions** (irritated skin, rash, postoperative wound, recent scar, presence of staples, traumatic wound, venous and arterial ulcer, diabetic wound, burns ...) as electrostimulation may aggravate cutaneous irritation or disrupt healing.
- In case of skin **sensitivity problems** in the lower abdomen or back, do not apply the patch to avoid a risk of secondary burns as a result of setting the intensity too high, because of the lack of sensitivity.
- Do not apply **on or near a cancerous or infected lesion** as the current produced may promote the spread of cancer cells to other parts of the body.
- This device is not intended and should not be used by a pregnant woman. 

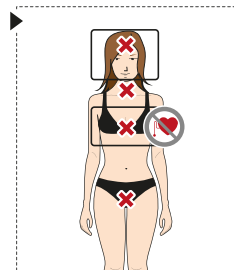### 4.3 For precaution ask for a medical or pharmaceutical advice

- In case of epilepsy or suspicion of epilepsy because electro stimulation could promote the occurrence of epileptic seizures.
- In case of heart problems because the electrical signals can interfere with the electrical signals of the heart.
- In case of fever over 39 °C because it could impact your watchfulness.

### 4.4 Precautions of use

- Do not use this device while driving (cars, machines...), or in case of activity where electrical stimulation or involuntary muscle contraction may be dangerous.
- Do not use this device while you sleep in order to avoid the risk of pain, inflammation, or burn due to excessive intensity adjustment that you would not be able to control for lack of alertness.
- Do not use this device in areas with staples, implants, or any metal parts in order to avoid the risk of burns or pain.
- Do not use this device in highly humid areas such as a bathroom or when you are bathing or showering.
- For hygienic reasons, do not share your adhesive gels with others.
- Keep out of children's reach and sight: they must not play with the device.

### 4.5 Adverse effects

- Light skin rash or irritation may occur in the area of application of the patch, this reaction is normal if temporal reaction. They are explained by the localized increase in blood flow. If you experience these symptoms, do not reapply the patch in the same area while the redness persists.
- Stop treatment immediately in case of skin irritations, burns or allergic reactions that may occur in the area of application of the device due to electrical stimulation or the adhesive gel.
- If you experience any other side effects, stop using the patch and report them to your doctor or pharmacist.

## 4.6 Interactions

- Do not use this device in the presence of electronic monitoring equipment (e. g. heart monitor, ECG alarm) or while performing an electrocardiogram or electroencephalogram. Electronic equipment may not operate properly when the patch is on.
- Be aware of the potential risk of burns if you take alcohol, analgesic or anti-inflammatory drugs at the same time, as they may affect your skin sensitivity. This is not a contraindication.

## 5. CLEANING

After use, if needed, the front side of the patch can be cleaned with a soft cloth slightly moistened:

- To clean the patch, make sure that the USB charging cable is unplugged.
- Do not use chemical detergents or abrasives. If the patch is very dirty, you might use a slightly soapy solution.
- Make sure that water does not get inside. If this should happen, do not use the device until it is completely dry.
- The back side of the patch, with adhesive gels, does not require cleaning.
- Cleaning and maintenance must be carried out under the supervision of a responsible adult if the user is mentally or physically impaired person.

## 6. STORAGE

- Keep this product away from moisture, high temperatures and direct sunlight, in its storage box.
- Keep the blue protective films to replace them on the patch after each use.

## 7. QUESTIONS & ANSWERS

Problems/Questions	Possible cause	Solution
The patch does not emit signal or does not start.	<ul style="list-style-type: none"> <li>The patch is totally discharged.</li> <li>The patch has an operating problem.</li> </ul>	<ul style="list-style-type: none"> <li>Charge the patch with the cable provided in the packaging.</li> <li>Contact the consumer service of URGO, in France: <a href="mailto:contact@urgo.fr">contact@urgo.fr</a> and in Belgium: <a href="mailto:contact@urgo.be">contact@urgo.be</a></li> </ul>
You do not feel any stimulation.	<ul style="list-style-type: none"> <li>The intensity level is too low.</li> <li>The patch is discharged.</li> <li>The gel needs to be re-humidified.</li> </ul>	<ul style="list-style-type: none"> <li>Increase the intensity level thanks to the "+" button.</li> <li>Charge the patch using the charging cable provided in the package and a CE-certified mains charger.</li> <li>Humidify the adhesive gels as indicated in section 3.2</li> </ul>

Problems/Questions	Possible cause	Solution
Electrodes give an unpleasant sensation.	<ul style="list-style-type: none"> <li>The intensity level is too high.</li> <li>The electrode adhesive gel does not stick enough or is damaged and doesn't have a sufficient contact with the skin.</li> </ul>	<ul style="list-style-type: none"> <li>Decrease the level of intensity with the button "⊖".</li> <li>Clean and dry your skin. If the patch do not correctly stick to your skin, humidify the gels as indicated in part 3.2.</li> <li>If the problem is persistent, change the adhesive gel and verify it covers all the electrical circuit/electrodes of the stimulator before use.</li> </ul>
One side of the patch sends stronger stimuli than the other.	<ul style="list-style-type: none"> <li>It is possible that every area of your body reacts differently.</li> <li>The patch does not adhere properly to the skin.</li> </ul>	<ul style="list-style-type: none"> <li>If this sensation is unpleasant, reposition the patch.</li> <li>Verify the good contact between the patch and your skin.</li> <li>Clean and dry your skin. If the patch does not stick correctly to your skin, humidify the gels as indicated in section 3.2.</li> <li>If the problem persists, change the adhesive gel and make sure it covers all the surface of electrical circuit of the stimulator before use.</li> </ul>
What should I do if the patch, without pressing any buttons, emits a series of beeps during the session?	<ul style="list-style-type: none"> <li>The patch does not properly adhere to the skin.</li> </ul>	<ul style="list-style-type: none"> <li>You must stick the patch again so the electrodes are completely in contact with the skin and start the program again by briefly pressing the "+" button.</li> <li>Your program will resume where it left off and you will need to adjust the intensity again.</li> </ul>
During the session I feel that the intensity decreases without touching the patch. Is this normal?	<p>Several causes are possible:</p> <ul style="list-style-type: none"> <li>The patch is not in contact with the skin.</li> <li>The patch is discharged.</li> <li>During the session you may get used to the intensity level, increase it with the "+" button.</li> <li>The gels must be moisten.</li> </ul>	<ul style="list-style-type: none"> <li>Verify the good contact between the patch and your skin.</li> <li>The battery can be almost discharged, recharge the patch using the cable provided.</li> <li>You can get used to the intensity during the session, increase it with the "+" button.</li> <li>Humidify the adhesive gels as indicated in section 3.2</li> </ul>
The patch does not adhere properly to the skin. What am I supposed to do?	<ul style="list-style-type: none"> <li>Cream or body milk may decrease the gels adhesiveness.</li> <li>The gels must be moisten.</li> </ul>	<ul style="list-style-type: none"> <li>First, clean and dry your skin.</li> <li>If your patch still does not adhere properly, moisten the gels as described in section 3.2.</li> <li>If the problem persists, replace the gels and check that they cover the entire electrical circuit of the electrostimulator before using it again.</li> </ul>

Questions	Answers
The treated area becomes red. Is it normal?	Temporary redness of the skin under the electrodes is benign, normal and it is explained by the localized increase of blood flow. You may have removed the patch too quickly from the skin and this could be causing redness as well. Try to remove the patch more gently next time. If skin irritation persists and you experience itching or irritation, an allergy may be the cause. Consult your doctor before using the device again.
Is it possible to apply the patch on clothing?	No, the program will be blocked because the electrical circuit is not closed: the clothes act as an insulating layer and prevent the circuit from closing. The patch (with adhesive gel) must be in direct contact with the skin.
Can I remove the patch without stopping it?	No, you have to turn off the patch first (press the "on/off" button), then after hearing 3 beep, remove the patch.
Can I use the patch to prevent the pain?	No. The patch does not prevent the pain. Use it from first symptoms.
Can I practice sports while wearing the patch?	You can walk and go about your daily activities while wearing the URGO ELECTROTHERAPY PAINFUL PERIODS RECHARGEABLE PATCH. As for practicing sports, as long as the patch remains firmly on your skin, it is not a problem, but if you sweat or move a lot, the adherence will probably decrease and so decrease or delete the patch's efficiency.
Can I use the patch if someone is wearing an implanted electronic or cardiac device?	Yes.
Can I apply the patch on a caesarean section scar?	You can apply the patch on a scar that is older than 6 months, if the scar is more recent than 6 months, we advise you not to apply the patch.
Will I feel the effects of the patch homogeneously even if I apply the patch on a scar?	Not necessarily, small superficial nerve fibers may have been damaged and interfere with sensations relating to the effects of the patch. Either way, don't place the patch on scars that are less than 6 months old.
How long after delivery can I use the patch?	You will be able to apply the patch after 8 weeks. But in the case of a caesarean scar less than 6 months old, we advise you not to apply the patch.
During my periods, I often feel more pain on one side than on the other, how should I apply the patch in this case?	In this case, the patch should be applied as close as possible to the painful area. However, even when placed in the center of the belly, the patch will continue to be effective for painful menstruation.
After sticking the gels on the electrical circuits, I can see small air bubbles. Is that bad?	No, the presence of small air bubbles on the electrical circuit does not interfere with the proper functioning of the patch. Make sure the gel is globally well stuck.
The adhesive gels were slightly damaged at the periphery (for example, when removing the transparent film): will the patch work less efficiently?	Even if the gels are slightly damaged at the periphery, the patch will work correctly provided that the electrical circuits are well covered.
Can I use the mains charger of my mobile phone?	Yes, if it is certified and delivers the current with the following output parameters: 5V and 0.3-2A (printed on the charger).
Can I use a fast charger?	No, because a fast charger is not compatible with the patch and may damage it.
Can I use my mobile phone cable?	No, use the cable supplied in the package.

## 8. DISPOSAL

For environmental reasons, do not dispose URGO Electrotherapy painful periods rechargeable patch in the household waste at the end of its lifespan. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive - WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

## 9. TECHNICAL SPECIFICATIONS

Model/type	URGO Electrotherapy Painful Periods Rechargeable patch
Power supply	Powered by internal 3.7V Li-ion battery
Waveform and wave shape	Biphasic rectangular waveform
Pulse duration	0 -200µs (Microseconds)
Pulse frequency	1 - 100 Hz ± 15%
Maximum voltage	Max. 16V+/- 20% (at 500 ohm load)
Treatment time	30 minutes
Output intensity	0 to 15 levels, adjustable
Programs	3 programs
Typical operation time of battery	About 240 minutes depending on the program used after full charge
Behaviour of the equipment while the rechargeable internal electrical power source is charging:	The indication light will blink during charging and will be still with full capacity
Typical service life of battery	300 times of recharging
Normal working ambient temperature	5°C to 40°C
Storage and transport ambient temperature	-25°C to +70°C
Atmospheric pressure for use and storage	70kPa-106kPa
Weight	30g
Automatic shut off	30 minutes
Degree of protection against electric shock	Type BF applied part
Type of protection against electric shock	Internally powered equipment
IP code	IP22: The device is protected against foreign object ≥ 12.5mm and from vertical falling water drops when device is tilted up to 15°
Number of reuse electrode adhesive gel	30 times
Expiry date of electrode adhesive gel	Indicated on electrode adhesive gel
Mode of operation	Continuous/discontinuous operation
Software version	AO
The time required for the equipment to warm from the minimum storage temperature between uses until it is ready for intended use	30 minutes
The time required for the equipment to cool from the maximum storage temperature between uses until it is ready for intended use	15 minutes
USB cable for charging	Please use output DC5V and output current 0.3-2.0A cable for charging
Normal working ambient humidity	15%RH-90%RH
Storage and transport ambient humidity	0%RH-90%RH
Electrode surface area	48cm <sup>2</sup>

**NOTE: NOT INTENDED TO BE STERILIZED.**

**Not for use in an OXYGEN RICH ENVIRONMENT  
For example, to use stimulator in a hyperbaric oxygen chamber or in an oxygen treatment environment is prohibited.**

This device complies with European standards EN60601-1, EN60601-1-2, EN60601-1-11 and EN60601-2-10 is subject to special precautions regarding electromagnetic compatibility. Note that portable and mobile RF communication equipment can affect this unit. More details can be requested from the stated Customer Service address.

The device meets the requirements of European Directive 93/42/EC concerning medical devices.

## 10. WARRANTY

Laboratoires URGO Healthcare commits to replace the defective device\* within 2 years from the date of purchase. The following are excluded from the guarantee:

- Damage caused by improper use: failure to comply with the instructions for use, use not in accordance with the original intended use, abnormal environmental conditions, unsuitable operating conditions, overloading or lack of maintenance or care.
- Damage caused by commercial use.
- Damage on the products that were purchased as second-hand products.
- Processed or modified products.

\*This excludes consumables (adhesive gels and other consumable electrotherapy accessories) when they are subject to normal wear and tear.

This warranty is in addition to the existing warranty obligations and does not affect your statutory rights as a consumer.

To claim the warranty, your request must be accompanied by a proof of purchase (receipt).

	Prohibited for persons wearing a pacemaker.		Type BF applied part: any device in contact with the patient, except devices for direct cardiac applications.
	Keep the product in a dry place.	<b>IP22</b>	IP code of the device.
	Manufacturer.		Authorised Representative in the European Community.
<b>CE 1639</b>	CE marking, Certificate issued by SGS Belgium.		Batch code.
	Date of manufacture.		Expiry date: date after which the product may not be used.
	Permissible temperature condition.		Product package should be recycled.
	CAUTION: read instructions for use (important precautions and warnings).		Read instructions for use.
	Humidity limitation.		Product subject to a sorting order for recycling.
	Keep away from sunlight.	<b>ROHS</b>	Restriction of the use of certain hazardous substances in electrical and electronic equipment.
	To protect the environment, the device and its battery must not be discarded with the household refuse at the end of its life. It must be disposed of via the existing collection points.		

Date of CE marking: 2021

Last revision of the notice: 02-2021

**JKH Health Co., Ltd.**

Address: 4-5F, Building 12, Hengmingzhu Industrial Park, Xinqiao  
Tongfuyu Industrial Area, Shajing, Baoan, Shenzhen, CHINA

Shanghai International Holding Corp. GmbH (Europe)  
Eiffestrasse 80, 20537 Hamburg, GERMANY

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